



CIDEZYME[®] XTRA

MULTI ENZYMATIC DETERGENT

Fast. Effective. Thorough.

CIDEZYME[®] XTRA Multi Enzymatic Detergent is uniquely formulated for automatic or manual endoscope reprocessing. Containing multiple enzymes, it provides rapid, more complete penetration and effective removal of bioburden.

EXPERIENCE THE BENEFITS

CIDEZYME[®] XTRA Multi Enzymatic Detergent is an efficient first step before high-level disinfection

- Advanced, low-foaming solution is biodegradable and rinses easily
- Contains a rust inhibitor to help protect valuable medical devices and surgical instruments
- Easily removes fats, lipids, proteins, starches and carbohydrates





CIDEZYME[®] XTRA

MULTI ENZYMATIC DETERGENT

FEATURES

MULTI-ENZYME FORMULATION

Cellulase, lipase, protease, amylase, and proprietary enzymes with advanced proteolytic action

LOW FOAMING

CORROSION INHIBITORS

BIODEGRADABLE

BENEFITS

Increased protein enzyme activity to provide a faster, more thorough penetration into hard-to-reach places

Effectively remove bioburden

Designed for use in automated systems and manual reprocessing

Ensures compatibility with medical devices

Safe for the environment

Materials Compatibility

CIDEZYME[®] XTRA MULTI ENZYMATIC DETERGENT HAS BEEN TESTED WITH A WIDE RANGE OF MATERIALS COMMON TO FLEXIBLE ENDOSCOPES INCLUDING:

Brass, copper, stainless steel, a variety of polymers, rubber, corrugated tubing and plastic

System Use

VALIDATED FOR USE IN THE EVOTECH[®] ENDOSCOPE CLEANER AND REPROCESSOR (ECR) DESIGNED FOR EITHER MANUAL OR AUTOMATIC REPROCESSING

Usage

FOR AUTOMATED SYSTEMS ½ oz per gallon of water (4 mL per 1 liter of water)

FOR MANUAL USE 1 oz per gallon of water (8 mL per 1 liter of water)

SHELF LIFE 18 months

STORAGE INFORMATION 59-86° F (15-30° C)

ORDERING INFORMATION

REORDER NO.	DESCRIPTION	CASE CONTENTS
22591	1 GALLON (3.8 L) container	2 X 1 GALLON (3.8L)

Please read and follow the CIDEZYME[®] XTRA Multi Enzymatic Detergent instructions for use prior to using this product for complete usage information, including contraindications and warnings.



FOR MORE INFORMATION CONTACT YOUR LOCAL ASP REPRESENTATIVE AT 1-888-783-7723 OR VISIT WWW.ASPJJ.COM

ADVANCED STERILIZATION PRODUCTS

Division of Ethicon US, LLC

a *Johnson & Johnson* company