

CIDEX® OPA Concentrate Solution Efficacy Against Human Coronavirus

Background

Coronaviruses are a family of enveloped positive-sense single-stranded RNA viruses, capable of rapid mutation and recombination. Coronaviruses are a common cause of mild upper respiratory infections, but they also include more serious illness like SARS and MERS, which can lead to severe disease. Coronaviruses are common in many mammals and, although rare, animal coronaviruses can infect humans and then spread between people¹.

The 2019 novel coronavirus, designated as SARS-CoV-2, is a new virus responsible for causing Coronavirus disease 2019 (COVID-19). It was first identified during an outbreak in Wuhan, Hubei Province, China in late 2019² and has since spread globally. COVID-19 has confirmed human-to-human transmission as well as demonstrated community transmission in the United States³. Although most patients with coronavirus have mild symptoms, the SARS-CoV-2 has demonstrated that it can escalate to a more severe form of the disease, which can include escalating to pneumonia. The outbreak was declared a Public Health Emergency of International Concern by the World Health Organization (WHO) on 30 January 2020⁴.

SARS-CoV-2 is spread primarily from person-to-person between people who are in close contact with one another (under 6 feet) and via respiratory droplets produced when an infected person coughs or sneezes¹. Spread and viral shedding has been shown to be possible prior to the onset of symptoms³. Lastly, it may be possible that COVID-19 can be spread by touching a surface of object that has the virus on it and then touching susceptible areas of the face, such as eyes, mouth and nose³. A recent study reported evidence that indicate the virus can survive on hard surfaces such as plastic and stainless steel for up to 72 hours and on cardboard for up to 24 hours⁵. SARS and MERS viruses can also survive on hard surfaces for greater than 48 hours⁶.

Efficacy against Coronavirus

Although ASP has not specifically conducted testing of CIDEX OPA Concentrate (OPA C) Solution against SARS-CoV-2, CIDEX OPA C Solution has been tested for efficacy against human coronavirus. Product development testing was performed using United States Environmental Protection Agency (EPA) Virucide Assay Methods⁷. In this development testing, preparations of viral samples of human coronavirus were inactivated by CIDEX OPA C Solution after a 5-minute exposure time at both room temperature† and at 50°C; which respectively achieved $\geq 4\text{-log}_{10}$ and $\geq 2.75\text{ log}_{10}$ reduction in virus titer. Testing was also completed using a diluted strength CIDEX OPA C Solution mixtures to present adequate challenge to CIDEX OPA C Solution. The results show that even at diluted concentrations, CIDEX OPA C Solution inactivates Human Coronavirus completely. These test results satisfy EPA requirements.

Based on this testing data regarding the efficacy of CIDEX OPA C Solution against coronaviruses, SARS-CoV-2 should be susceptible to CIDEX OPA C Solution.

Additionally, coronaviruses such as SARS-CoV-2 are lipid viruses (enveloped viruses). Lipid viruses are typically susceptible to various low and medium disinfection modalities according to the guide of the hierarchy of biocide resistance published by the Centers for Disease Control and Prevention (CDC)⁸. CIDEX OPA C Solution is a High-Level-Disinfectant and has been shown to be effective against microorganisms such as mycobacteria and polio virus, which are microorganisms that typically exhibit greater resistance to disinfection modalities than lipid viruses like the coronavirus. Based on what is currently known about microorganism resistance to disinfection, CIDEX OPA C Solution would be efficacious against lipid viruses such as coronavirus.

Please note that per the CIDEX OPA C Solution Instructions For Use (IFU), CIDEX OPA C Solution is only intended to be used with the ASP EVOTECH® Endoscope Cleaner and Reprocessor (ECR), which is an Automated Endoscope Reprocessor (AER). CIDEX OPA C Solution IS NOT INTENDED FOR MANUAL USE. During normal usage, a pre-programmed disinfection cycle of a minimum of 5 minutes exposure time at 50°C (122°F) in CIDEX OPA C Solution is required for high-level disinfection (HLD). It is important to properly follow the CIDEX OPA C Solution IFU for detailed directions when processing any instrument using CIDEX OPA C Solution, especially the pre-cleaning

of all instruments and the wearing of appropriate Personal Protective Equipment (PPE) when handling OPA C solutions. HLD guidelines in your facilities as well as the HLD instructions of manufacturers should also be followed.

Please keep in mind to follow the safety precautions of the local health authorities of your country as well as health care facility when dealing with infected patients.

For any additional questions regarding SARS-CoV-2 or Coronaviruses in relation to our products, please submit a Medical Information Request by visiting www.asp.com.

† Note that this parameter was used during development testing, when performing actual reprocessing of medical devices or equipment, the required exposure time and temperature are 5 minutes at 50°C (122°F) in the ASP EVOTECH ECR.

Sincerely,

ASP Medical Affairs
Advanced Sterilization Products

33 Technology Drive.

Irvine, CA 92618

Email: Medical@ASP.com

www.ASP.com

REFERENCES:

1. Q&A on coronaviruses (COVID-19). <https://www.who.int/news-room/q-a-detail/q-a-coronaviruses#>. Accessed 03/23/20.
2. Chan JF-W, Yuan S, Kok K-H, et al. *A familial cluster of pneumonia associated with the 2019 novel coronavirus indicating person-to-person transmission: a study of a family cluster*. Lancet. 2020; (published online Jan 24). [https://doi.org/10.1016/S0140-6736\(20\)30154-9](https://doi.org/10.1016/S0140-6736(20)30154-9)
3. CDC Coronavirus Disease 2019 (COVID-19) Situation Summary. <https://www.cdc.gov/coronavirus/2019-ncov/prepare/transmission.html>. Accessed 03/23/20.
4. WHO Rolling updates on coronavirus disease (COVID-19). <https://www.who.int/emergencies/diseases/novel-coronavirus-2019/events-as-they-happen>. Accessed 03/23/20.
5. Neeltje van Doremalen, Trenton Bushmaker, Dylan H. Morris, Myndi G. Holbrook, Amandine Gamble, Brandi N. Williamson, Azaibi Tamin, Jennifer L. Harcourt, Natalie J. Thornburg, Susan I. Gerber, James O. Lloyd-Smith, Emmie de Wit, Vincent J. Munster. *Aerosol and Surface Stability of SARS-CoV-2 as Compared with SARS-CoV-1*. New England Journal of Medicine, 2020; DOI: 10.1056/NEJMc2004973. <https://www.nejm.org/doi/full/10.1056/NEJMc2004973>. Accessed 03/23/20.
6. Otter JA, Donskey C, Yezli S, Douthwaite S, Goldenberg SD, Weber DJ. Transmission of SARS and MERS coronaviruses and influenza virus in healthcare settings: the possible role of dry surface contamination. J Hosp Infect. 2016;92:235–250. doi: 10.1016/j.jhin.2015.08.027. [https://www.journalofhospitalinfection.com/article/S0195-6701\(15\)00367-9/fulltext](https://www.journalofhospitalinfection.com/article/S0195-6701(15)00367-9/fulltext). Accessed 03/23/20.
7. EPA Pesticide Assessment Guidelines: Subdivision G, Product Performance.
8. Decreasing order of resistance of microorganisms to disinfection and sterilization and the level of disinfection or sterilization. <https://www.cdc.gov/infectioncontrol/guidelines/disinfection/tables/figure1.html>. Accessed 01/28/20.

DISCLAIMER

Important information: Prior to use, refer to the instructions for use supplied with this device for indications, contraindications, warnings and precautions.

ADVERSE EVENT REPORTING

Remember to contact our local representative in case any adverse event or product quality complaint occur while using our product and comply with your local regulations in terms of notification.

*EVOTECH® Endoscope Cleaner and Reprocessor (ECR) and ENDOCLENS-NSX™ Automated Endoscope Reprocessor (AER) are not CE marked devices and therefore not available in all markets